Cani-EX

Canine SEMEN EXTENDER

CANI-EX, a canine semen extender for fresh use and cooled transport.

Instructions for use.

Have beforehand enough water (distilled or demineralized water) stored in an incubator that is set at 30°C. For best results, prepare the CANI-EX extender just before collecting the semen. Prepare 250 ml water at 30°C in a glass beaker or bottle. Add the CANI-EX powder to the water. Stir the solution slowly until the powder is completely dissolved. Then put the CANI-EX extender in the incubator (set at 30°C) until use.

- At the moment of use, the temperature of the CANI-EX extender should be at the same temperature as the semen
- If the ejaculate is contaminated with prostatic fluid, a washing procedure is recommended. Measure or weigh the volume of the entire ejaculate. Dilute the ejaculate with the CANI-EX extender (1:3 V/V): meaning one part of semen and 3 parts of CANI-EX. Centrifuge for 10-15 minutes at 700 g. Remove the supernatant with a pipet and bring the volume back to the original volume of the ejaculate with CANI-EX.
- Measure the semen for concentration, motility, progressive motility and if need morphology and keep record of the obtained results.
- Add the amount CANI-EX extender needed to obtain the correct concentration. Usually, and based on the semen evaluation, a ratio of one part semen with 3 to 5 parts of CANI-EX extender are used to make the appropriate dilution.

Immediately after dilution, the tubes with diluted semen may be passively cooled (in the dark) at room temperature or actively cooled in a controlled climate box set at 15-17°C for about an hour. Put the tubes with diluted semen thereafter in a refrigerator, preferably on a rocking table.

Unused CANI-EX medium can be stored in the refrigerator for later use, but it is recommended to use it within 5 days.

Freezing of smaller quantities of CANI-EX extender.

If you often need small quantities of extender (for example 12 to 15 ml), you may freeze portions of the extender in plastic tubes. The tubes can then be stored in the freezer until use. Just before collecting the semen you can thaw the desired quantity of extender.

In that case, dissolving the CANI-EX powder can then best be done in 250 ml of water with a temperature of 20°C. After dissolving the powder, wait 15 minutes, then stir or shake one last time and finally fill the plastic tubes (15 ml) with the CANI-EX extender and store in the freezer at – 20°C.

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